

FAQ's - Model JJT889 - No-Touch Infrared Thermometer

The thermometer keeps showing "Lo". What does that mean?

The thermometer is indicating the measured temperature is lower than 89.6 degrees Fahrenheit (32 degrees Celsius) when the unit is in Human Temperature setting. (You can press and hold + to switch between Human Temperature or Household Item Temperature settings.)

Also, the thermometer might be too far away from the source. Please move the thermometer closer to the source you'd like to measure and try again.

Aim the thermometer at the subject whose temperature you want to measure, staying within about 2"-6" of the surface of the subject. Press the Temperature Trigger on the front of the thermometer and the temperature will immediately be displayed on the screen.

Note: "Lo" does *not* mean low battery, it means low temperature, as explained above.



"Lo" indicates a low temperature.

How can I use the thermometer?

1. How to Power ON:
 - I. Press the Temperature Trigger and the thermometer will power ON.
 - II. Wait approximately 3 seconds and press the trigger again to test the thermometer. (Must be within approximately 2"-6" of the subject for the sensor to register a measurement.)
2. **SET**: Press to turn volume ON/OFF. (The volume indicator will appear on the LCD when the volume is ON and will disappear when volume is OFF.) Press and hold to access previous temperature readings list. (Thermometer can store the 16 most recent previous temperatures taken.)
3. **M**: Press to change temperature reading units between Celsius and Fahrenheit.
4. **+/-** : When in the previous temperature reading list (press and hold **SET** button to access previous temperature readings list) press to scroll up and down the list of saved temperatures. Press and hold + or - to clear the previous temperature reading list. Press and hold + to switch between Human Temperature or Household Item Temperature settings.

I installed new batteries, but the battery icon shows partially filled battery symbol. What should I do?

A partially filled battery symbol will show on the LCD. This is the software design. Even when the battery is full, the battery indicator will always display what appears to be a partially filled battery symbol.



Even when the battery is full, the battery indicator will show partially filled battery symbol. This is normal.

Do I need to calibrate the thermometer for accurate readings?

DO **NOT** ATTEMPT TO CALIBRATE THE UNIT. THIS UNIT HAS BEEN CALIBRATED BY THE FACTORY.

How can I change from Human Temperature setting to Household Item Temperature setting?

Press and hold + to switch between Human Temperature or Household Item Temperature settings.